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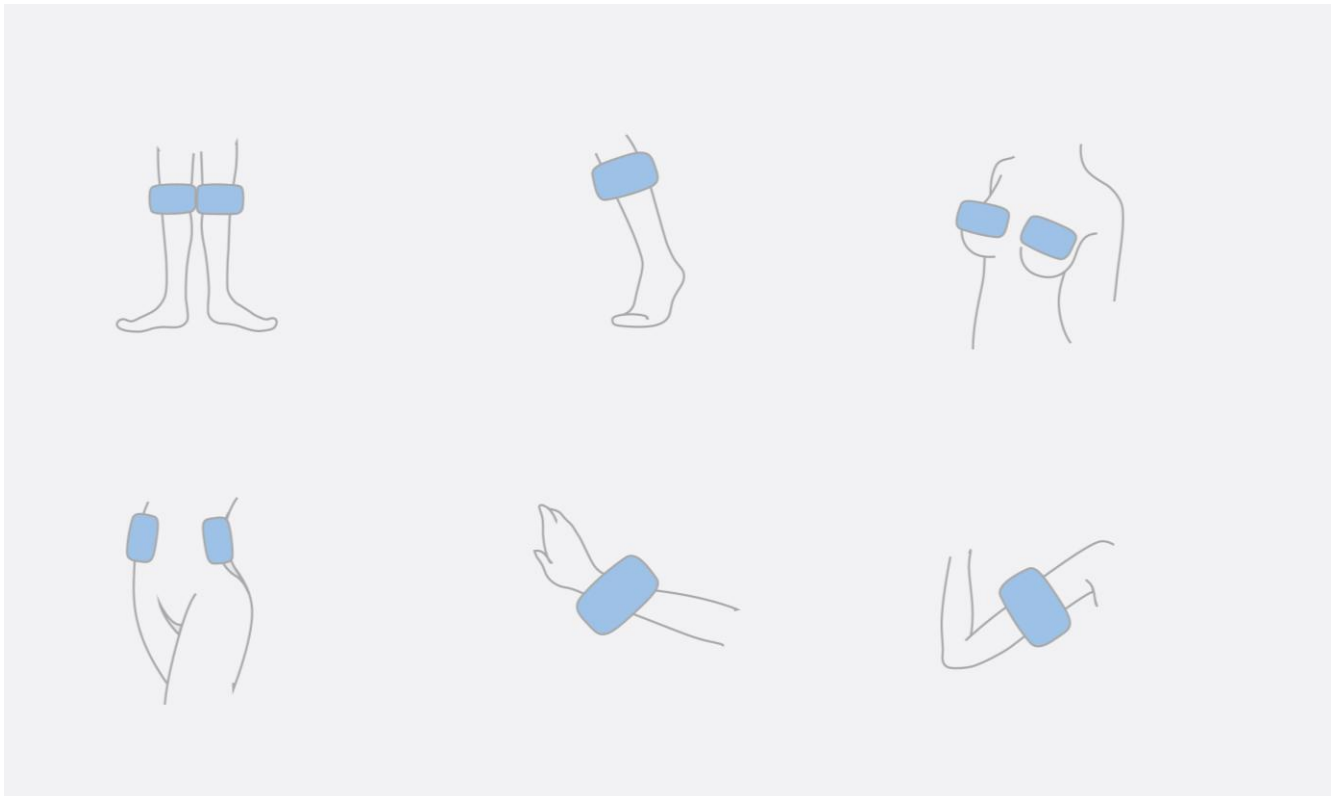
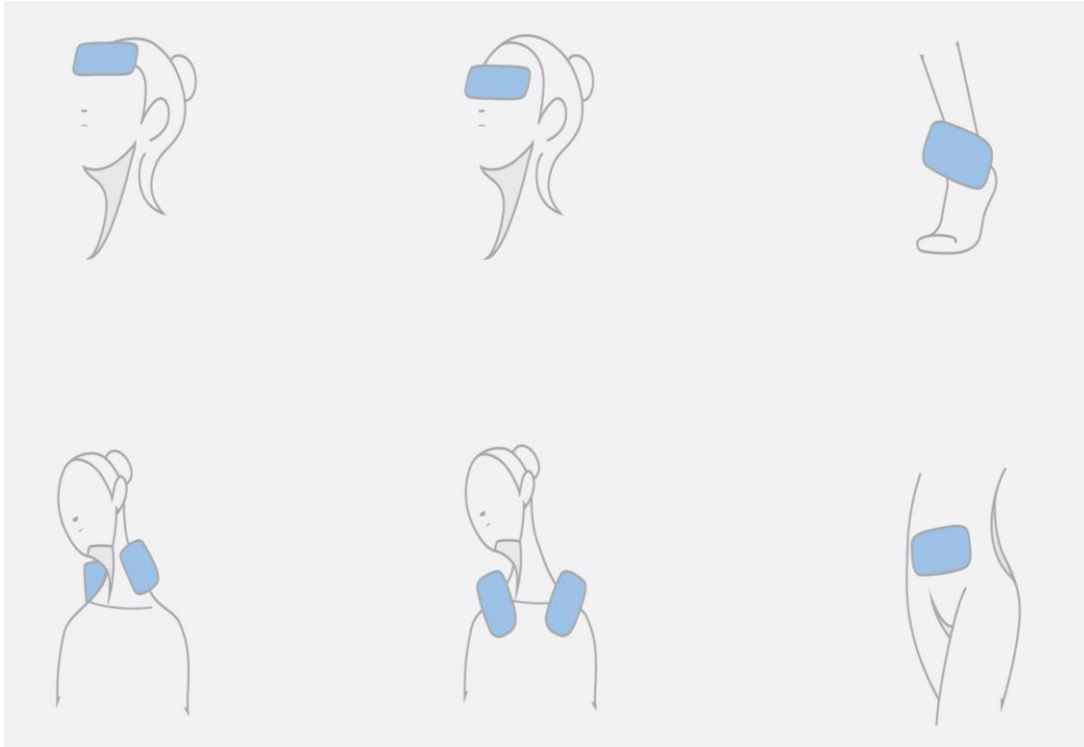
B. (Hot compress/Microwave)

1. Make sure the inner dish of the microwave is clean
2. Lay flat the pad on the inner dish and heat up for 30 sec. in a 500W microwave. (Heating time may vary according to different microwave power)
3. May increase the desired temperature by reheating the pad for no longer than 5 seconds per episode.

C. (Cold compress) Store the pad in the freezer before use

1. Put the pad in the freezer for 20 minutes, or
2. In ice water for 30 minutes, or
3. In a fridge for 1 hour.
4. Remove the pad from the freezer or cold water and wrap it with a cloth or insert it in the Chloson Insulate Cover for better result.
5. Store the pad in a clean bag when not in use.

*Duration of use:

1. Effective cold compress time: 30 minute
2. Suitable cold compress temperature: 3°C~15°C
3. Effective hot compress time: 15~30 minute
4. Suitable hot compress temperature: 40°C~55°C

***Caution:**

1. In order to avoid burns and frostbite, do not use the pad on the same area for too long.
- 2 Do not use this product if there is skin injury.
- 3 Do not over compress the product or use sharp object to pinch the pad surface.
- 4 If the gel leaks out and contaminate clothing, wipe it with wet cloth and wash it with water.
- 5 Do not use the product in any way that is not purposely designed for
- 6 Avoid storage in high temperature, moist or direct sunlight environments
- 7 If the substance leaks out and touches the body or eyes, flush thoroughly with warm water and seek medical attention immediately.
- 8 Seek medical attention if accidental swallowing does occur.
- 9 Keep out of reach for children and use under adult guidance.

10. The content should be disposed according to plastic waste classification.

***Materials:**

Outer Material: Nylon + PE

Contents: Non-toxic gel

*Permit number: Class I Medical device
No. 004393

*Shelf life: 5 years

*Made in Taiwan

*Lot no.: Same as Manufacture date

*Date of manufacture: as marked on the
box

*Company Name: Chloson Co., Ltd.

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